
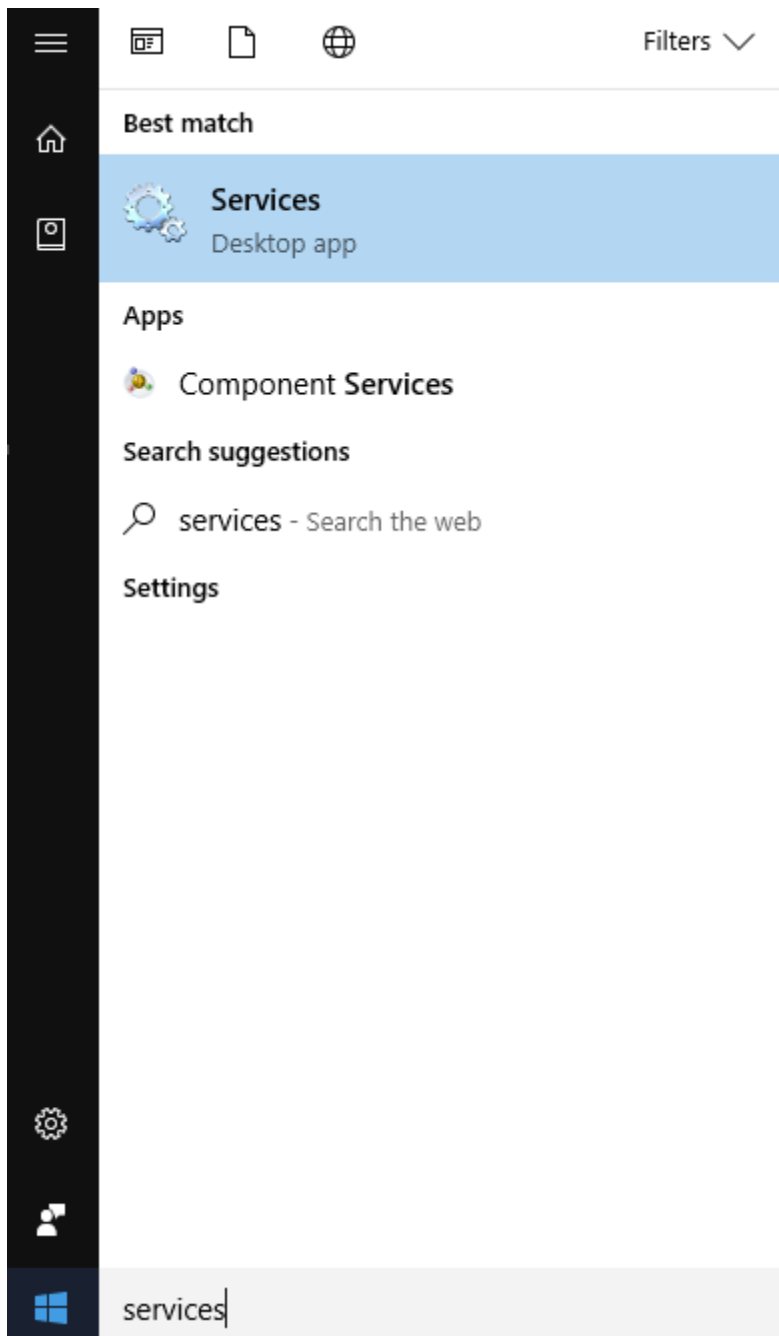



Panoramic Windows 10 Computer blue screen fix:

- 1) Power off the Panoramic machine or unplug the data cable from the Panoramic machine to the back of the computer.
- 2) Re-start your computer.
- 3) Once the computer starts back up, click the windows button () at the bottom left of your screen and type "services".




4) Click the silver gear icon.

This will open the Services window. Scroll down until you find Windows Update (the services are listed in alphabetical order):

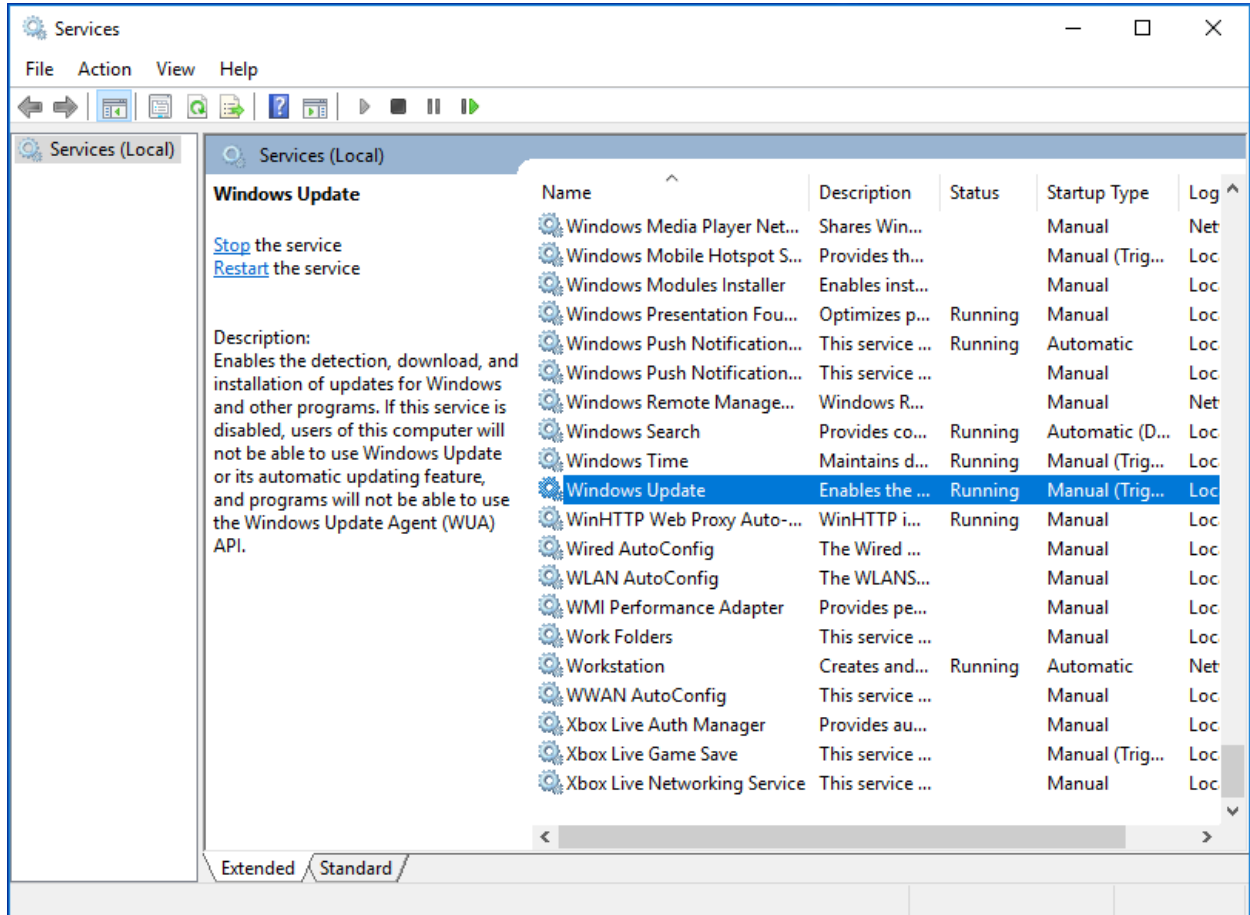


5) Right-click Windows Update and select properties:

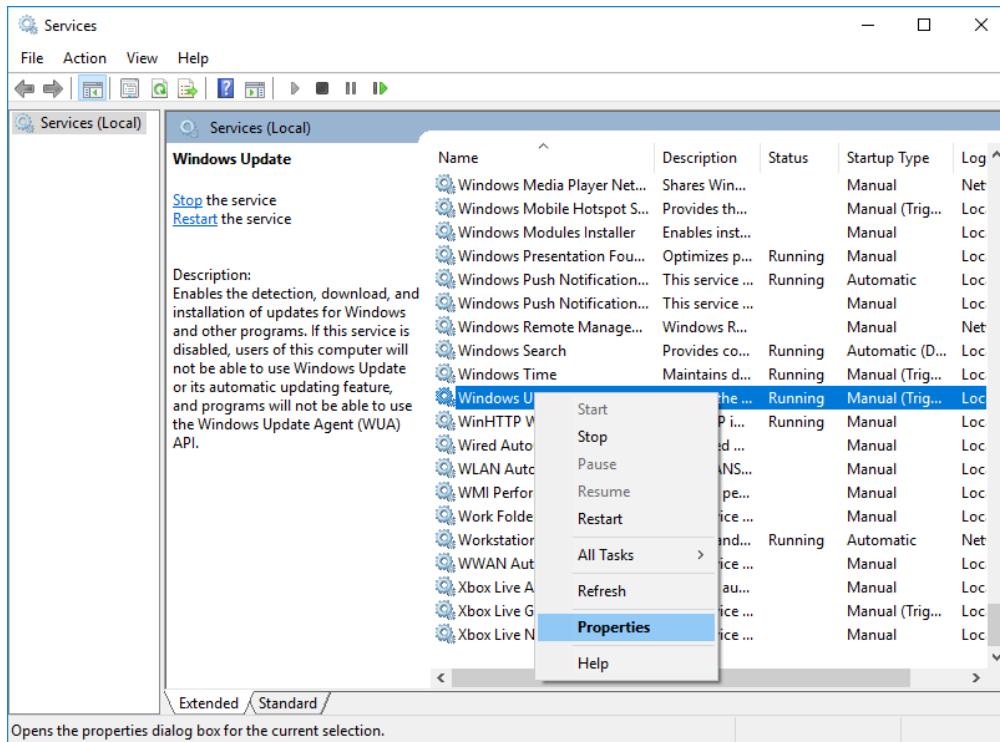


This will open the properties window.

6) Press the Stop button.

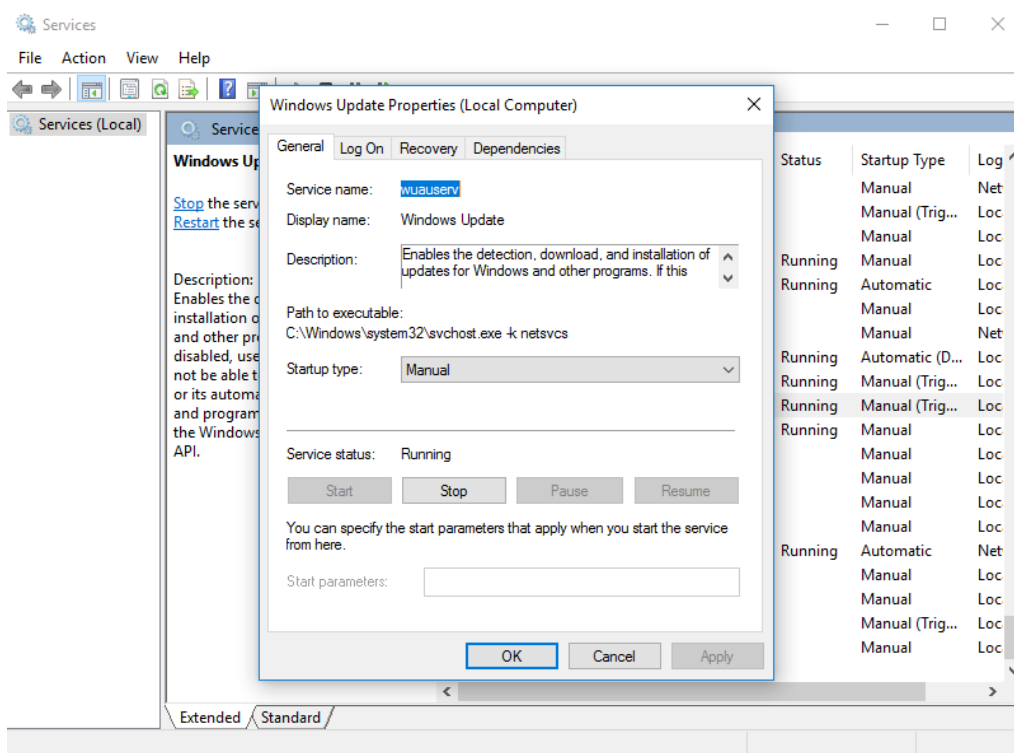

7) Click the Startup type: dropdown, select Disabled, and click OK.

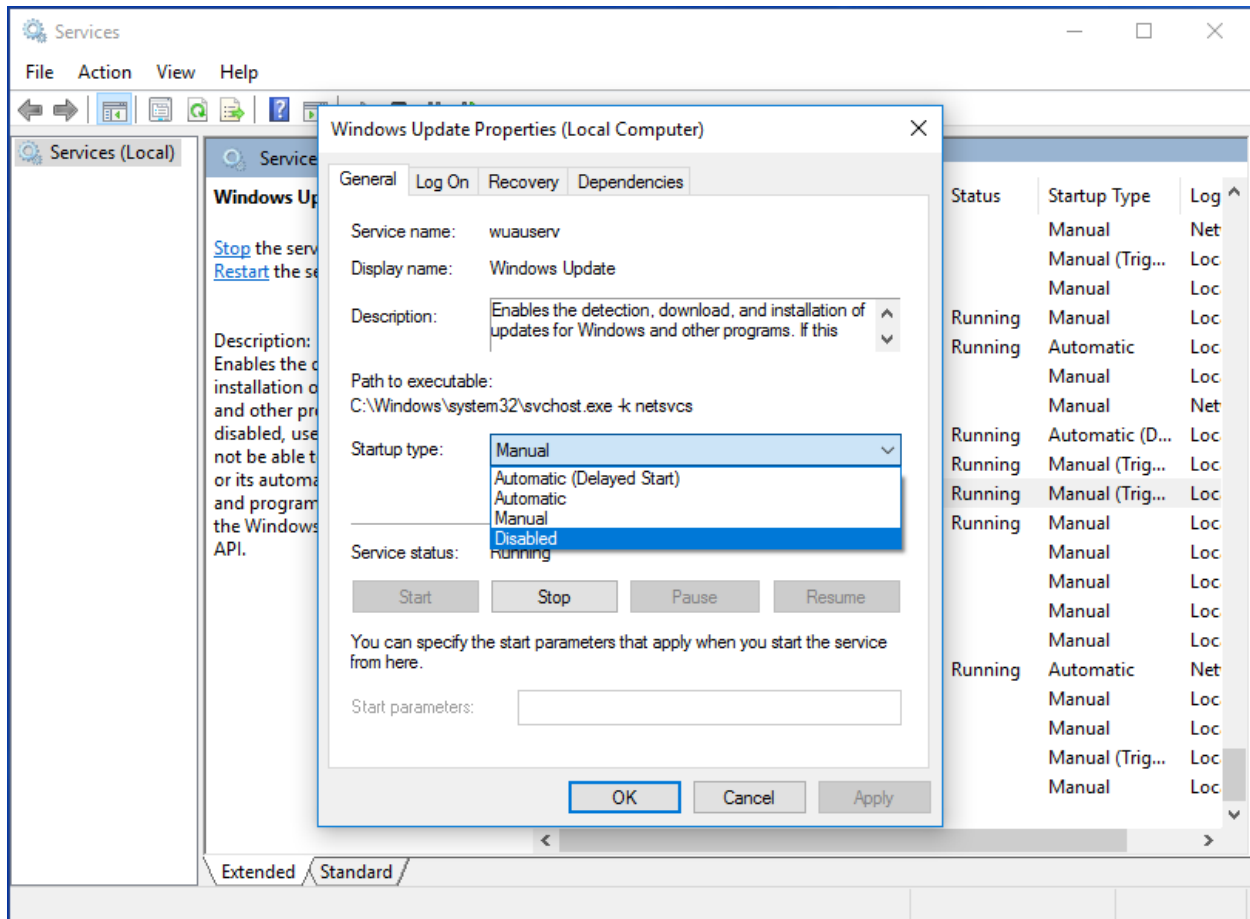

8) Reboot the computer and it should be safe to power the Panoramic machine back on, or plug it back in.

If you have completed the steps above and continue to get the blue screen, please e-mail your contact information (name, phone number, e-mail, serial number) to:

support@pancorp.com